1

1 PROPER CHARGING

For as long a life cycle of the battery as possible, return the light to the charging base as often as possible – e.g. after each patient.

2 CLEANING

Battery contacts should be cleaned regularly to ensure that they are capable of conducting and therefore charging properly – e.g. by means of mandatory wipe disinfection of the light before each use.

3 STORAGE / PRACTICE HOLIDAYS

In order to prevent premature wear, always charge the battery and do not store batteries for longer than 6 months.

4 PROTECT FROM EXTREME TEMPERATURES

For full operational reliability, always use (and store) the battery at room temperature or between 15 °C (59 °F) and 30 °C (86 °F).

5 REGULAR REPLACEMENT

Since the battery is a consumable, it has to be replaced if the performance drops considerably or after its typical life cycle has expired after approx. 2½ years.

6 DO NOT PURCHASE BATTERIES IN ADVANCE

In order to avoid irrevocable deep discharge if the battery is not used, a new battery is best purchased only once the old LiPo battery needs to be replaced.

7 NO DISPLAY DURING CHARGING

Check if the handpiece has been correctly placed into the charging base. Clean battery contacts. Information: "Charging Battery" briefly appears in the display before the display returns to the stand-by mode.

If cleaning the battery contacts does not improve the results, remove the battery from the apparatus. Charge the battery separately from the handpiece in the charging base for approx. 10 minutes. If the error remains, please contact your dealer or your local Service Center.

8 SAVETY NOTE

Please note that the lithium polymer battery may react with explosion, fire, and smoke development in case of inappropriate handling and mechanical damage.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.