# VivaStyle®

<b>EN</b> <b>Instructions for Use for at-home applications</b> – Tooth whitening gel	<b>FI</b> <b>Käyttöohje kotikäyttöön</b> – Hampaiden valkaisugeeli
<b>DE</b> <b>Gebrauchsinformation für die Behandlung zu Hause</b> – Gel zur Zahnaufhellung	<b>NO</b> <b>Bruksanvisning for anvendelse hjemme</b> – Gel til bleking av tenner
<b>FR</b> <b>Mode d'emploi pour le traitement à domicile</b> – Gel de blanchiment dentaire	<b>NL</b> <b>Gebruiksaanwijzing voor toepassing thuis</b> – Gel voor het witter maken van tanden
<b>IT</b> <b>Istruzioni d'uso per il trattamento domiciliare</b> – Gel per lo sbiancamento	<b>EL</b> <b>Οδηγίες χρήσης για εφαρμογή στο σπίτι</b> – Ζελέ λεύκανσης δοντιών
<b>ES</b> <b>Instrucciones de uso para la aplicación en el domicilio</b> – Gel de blanqueamiento dental	<b>TR</b> <b>Evde kullanım için Kullanma Talimatı</b> – Diş beyazlatma jeli
<b>PT</b> <b>Instruções de Uso para aplicações domésticas</b> – Gel para clareamento dental	<b>RU</b> <b>Инструкция для применения пациентом в домашних условиях</b> – гель для отбеливания зубов
<b>SV</b> <b>Bruksanvisning för hemmabehandling</b> – Gel för tandblekning	<b>PL</b> <b>Instrukcja stosowania materiału wybielającego w warunkach domowych</b> – Żel do wybielania zębów
<b>DA</b> <b>Brugsanvisning til hjemmeapplicering</b> – Gel til tandblegning	

For dental use only.

**Date information prepared**  
2014-10-06/WE3/Rev.1

**Manufactured for**  
Ivoclar Vivadent AG  
FL-9494 Schaan/Liechtenstein  
[www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)

**ivoclar**  
**vivadent**  
*clinical*

## English

### Description

VivaStyle is a ready-for-use whitening gel for lightening discoloured teeth. The product is administered by the user. The gel is applied by means of a custom-fitted tray.

### Composition

Glycerin, Urea (Carbamide) Peroxide, Carbomer, Tromethamine, Potassium Nitrate, Disodium Pyrophosphate, PEG-40 Hydrogenated Castor Oil, Sodium Stannate, Dipotassium EDTA, Aroma.

Active ingredients: 10% or 16% carbamide peroxide, which is equivalent to 3.6% and 5.8% hydrogen peroxide respectively.

One syringe contains 3.7 g gel. This corresponds to 3 ml.

### Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

### Contraindication

VivaStyle Gel should not be used

- during pregnancy or lactation;
- by children;
- by heavy smokers;
- by people who consume large quantities of alcohol;
- by people who are known to be allergic to any of the material's ingredients;
- in the presence of cervically exposed teeth;
- in the presence of untreated primary or secondary caries;
- in the presence of restorations with marginal gaps.

Applicable to EU countries:

- VivaStyle is not intended for use by patients under the age of 18
- For sale to dentists only

### Side effects

Temporary sensitivity of the oral mucous membrane or the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

### Application

- After a dental examination has been carried out and the duration of the treatment has been determined, VivaStyle 10% or VivaStyle 16% is administered by the patient at home as instructed by the dental professional.
- The first application of any tooth whitening cycle should be performed by dentists or under their supervision, provided the same level of safety is guaranteed. Subsequently, the gel must be placed at the disposal of the patient to complete the bleaching cycle at home.
- Before VivaStyle 10% or VivaStyle 16% is used, the dental professional should carefully examine the patient's medical history, inform the patient about the application procedure and clean the patient's teeth professionally. It is advisable for the dental professional to monitor the whitening process.

### Step-by-step instructions

1. Prior to the application of the VivaStyle gel, thoroughly brush and floss your teeth.
2. Remove the cap from the tip of the syringe. Place a small amount of gel toward the facial side of each tooth in the tray. Use the calibration markings on the syringe as a guide when dispensing the material. Do not overfill the gel in the trays. Place the cap back on the application tip.
3. Place the gel-filled trays over your teeth. Do not squeeze the tray when you insert it.
4. Use a toothbrush, cotton bud or index finger to remove excess gel. Do not swallow.
5. Wear the tray as prescribed by your dentist, e.g. for one hour a day.
6. Remove the tray after the prescribed time.
7. Rinse the mouth thoroughly to remove residual gel. Spit everything out – do not swallow.
8. Clean the tray with a toothbrush under running water. Always store the tray in the accompanying tray case.
9. Repeat the treatment with the VivaStyle gel every day if possible or according to the schedule prescribed by your dentist.

### Notes

- Overfilling the tray does not result in faster or better results. Instead, it may irritate the gums.
- Do not eat, drink or smoke while wearing the tray.
- Until you have finished the full treatment
  - avoid consuming citrus fruits and fruit juices, as they can promote hypersensitivity;
  - avoid, or at least try to minimize, the consumption of coffee, tea, red wine and colas as well smoking to prevent the risk of staining the teeth.
- The gel only whitens natural teeth. After the treatment, the appearance of existing restorations in the front teeth may no longer match that of the whitened teeth. Therefore, these restorations may have to be replaced.
- The treatment with VivaStyle 10% or VivaStyle 16% should be completed at least two weeks prior to the placement of tooth-coloured restorations.
- The teeth cannot be made to look whiter than their natural colour.

If you have any questions about the treatment or if you experience uncomfortable sensitivity, please contact your dental office.

#### **Warning**

Avoid contact of the material with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

#### **Storage**

- Protect VivaStyle syringes from heat and sunlight
- Do not use after the date of expiration
- Store at 2–28 °C / 36–82 °F.

#### **Keep out of the reach of children!**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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