

# SR Ivocap® Elastomer

The special elastomer-based resin



# SR Ivocap® Elastomer

**SR Ivocap® Elastomer** – special elastomer-based resin. The material is processed using the SR Ivocap technique and is suitable for the following applications:

- Positioners
- Mouth guards for athletes such as boxers, ice hockey players, etc
- Bruxism splints
- Bite splints requiring elasticity

## Advantages

- Highly accurate fit
- Comfortable to wear
- Neutral taste and smell

## Instructions for Use

- The material is processed using the SR Ivocap technique exclusively
- Always use two injection channels
- Allow the resin to rest for ten minutes after injection
- Polymerization time of 45 minutes at 95–100°C
- Store the material in a cool, dark place

## Delivery forms

- 10 SR Ivocap Elastomer capsules containing 20 g of polymer
- 10 SR Ivocap Elastomer capsules containing 16.5 g of monomer

## Sample application

Fabrication of a positioner



Duplicate model



Modelled mouth guard



Investment in the SR Ivocap flask



Set-up



Examples of finished pieces



This product forms part of our Removable Prosthetics category. All the products of this category are optimally coordinated with each other.

**Ivoclar Vivadent AG**  
Benderstr. 2  
9494 Schaan  
Liechtenstein  
Tel. +423 235 35 35  
Fax +423 235 33 60  
www.ivoclarvivadent.com

Descriptions and data constitute  
no warranty of attributes.  
© Ivoclar Vivadent AG, Schaan/Liechtenstein  
562769/en/2016-03

**ivoclar**  
**vivadent**  
passion vision innovation