


Service+ | Implant Prosthetics¹ | Step-by-Step 3shape


Customized Abutment

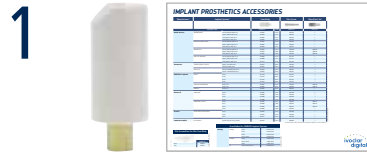

Use the suitable scan body
(see: Implant Prosthetics Accessories)



Implant selection
 Category: Ivoclar Digital Customized Abutment
 System: Select the desired implant system
 Kit: Select diameter

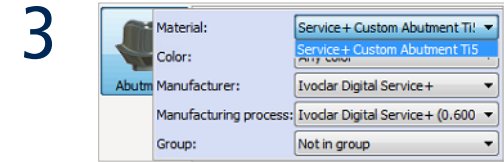

Material selection
 Material: Service+ Custom Abutment T15
 Color: polished or unpolished emergence profile
 Manufacturer: Ivoclar Digital Service+


Hybrid Abutment²



Use the suitable scan body
(see: Implant Prosthetics Accessories)




Implant selection
 Category: Ivoclar Digital Hybrid-Abutment-Crown
 System: Select the desired implant system
 Kit: Select diameter

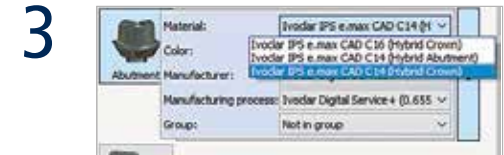

Material selection
 Material: Select Ivoclar material
 Color: Select translucency level and shade
 Manufacturer: Ivoclar Digital Service+


Hybrid Abutment (CAMLOG)

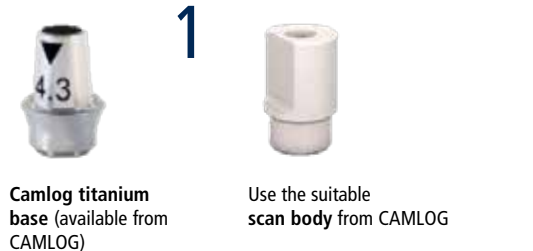

Use the suitable scan body from CAMLOG



Implant selection
 Category: Ivoclar Digital Hybrid Abutment (CAMLOG)
 System: Select the desired implant system
 Kit: Select diameter

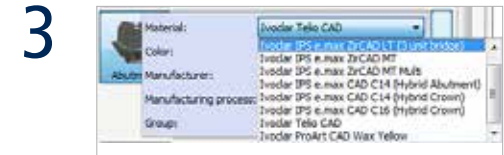

Material selection
 Material: Select Ivoclar material
 Color: Select translucency level and shade
 Manufacturer: Ivoclar Digital Service+


¹ Requirements: 3Shape (version 16.1 or higher) Abutment Designer or Implant Bar and Bridge or CAD Points, registration with Ivoclar Digital Service+, import of the library
² also possible with titanium bases from Medentika (2nd generation). Requirements: Installation of the Medentika library and link to Ivoclar materials.

Service+ | Implant Prosthetics¹ | Step-by-Step 3 shape

Implant Bar/Bridge



Use the suitable **scan body**
(see: Implant Prosthetics Accessories)

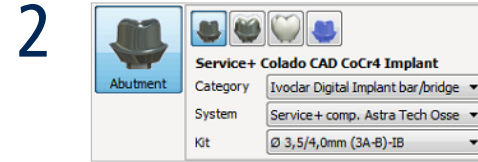


Implant selection
Category: Ivoclar Digital Implant bar/bridge
System: Select the desired implant system
Kit: Select diameter



Material selection
Material: Service+ Colado® CAD Ti5 Implant or Service+ Colado® CAD CoCr4 Implant
Color: polished or unpolished emergence profile
Manufacturer: Ivoclar Digital Service+


Gingiva Former

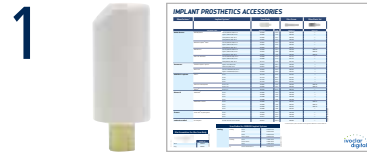

Use the suitable **scan body**
(see: Implant Prosthetics Accessories)



Implant selection
Category: Ivoclar Digital Gingiva former
System: Select the desired implant system
Kit: Select diameter

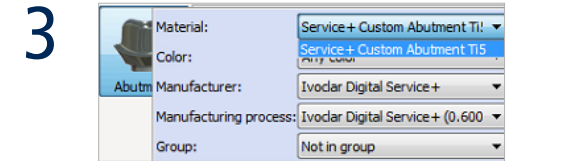


Material selection
Material: Service+ Custom Abutment Ti5
Color: polished or unpolished emergence profile
Manufacturer: Ivoclar Digital Service+

Hybrid Bridge³



Camlog titanium base³
Use the suitable **scan body** from CAMLOG

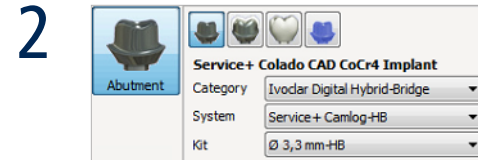

Implant selection
Category: Ivoclar Digital Hybrid-Bridge
System: Select the desired implant system
Kit: Select diameter



Material selection
Material: Select Ivoclar material
Color: Select translucency level and shade
Manufacturer: Ivoclar Digital Service+

³ also possible with titanium bases from Medentika (2nd generation). Requirements: Installation of the Medentika library and link to Ivoclar materials.