# Digital Shade Assistant

Instructions for Use | Bedienungsanleitung | Mode d'emploi | Istruzioni d'uso  
Instrucciones de uso | Instruções Operacionais

# Digital Shade Assistant

## Instructions for Use

Certain Programat® furnaces are equipped with a function to determine the tooth shade based on three selected shade guide reference teeth. The process of shade determination is divided into two parts:

- Determining the reference teeth and imaging involving the patient (clinical part)
- Evaluating the image data by means of the DSA function directly with the Programat

These Operating Instructions describe the **clinical** process. The evaluation of the image data using the Digital Shade Assistant (DSA) function is described in the Operating Instructions of the respective Programat furnace.

To determine the three reference teeth and imaging, please proceed as follows:

### 1. Placing an OptraGate®

Place an OptraGate® in the patient's oral cavity so that the tooth neck is also visible and shadows on the incisors are prevented. The OptraGate facilitates shade determination. However, its use is not mandatory. During shade determination, make sure that the teeth are moist and not dehydrated.

### 2. Determining the reference teeth

Now, the three most similar shade guide teeth are selected.

The Digital Shade Assistant system is exclusively compatible with the following shade guides:

- Ivoclar Vivadent A–D shade guide
- Ivoclar Vivadent Chromascop shade guide

### 3. Loading the Digital Shade Assistant holder

Position the three selected reference teeth either with the tooth neck facing upwards or downwards in the DSA holder.

- The orientation of the reference teeth "incisal edge to incisal edge" or "incisal edge to tooth neck" must be the same for all three reference teeth.
- The orientation "incisal edge to incisal edge" of the reference teeth is only possible with software version V2.0 and higher.

#### 4. Positioning the DSA holder at the patient

- Position the three reference teeth as closely as possible to the tooth to be analyzed
- Position the teeth on the same level as much as possible (see sketch)
- Avoid reflections on the teeth
- We recommend soliciting the help of a second person when positioning the DSA holder

#### 5. Selecting the adequate image section

Selecting the proper image section must be given special attention. The following elements must be well visible in the centre of the picture:

- Complete tooth to be analyzed
- Three reference teeth and the imprint on the pin of the shade guide teeth
- Markings (◊ I) in the upper corners of the DSA holder

#### 6. Taking the picture

Now, a digital picture can be taken.

#### 7. Picture transfer

The digital picture in .jpg format can be transferred to the furnace:

- By e-mail
- By USB flash drive or SD memory card
- Via PrograBase directly to the Programat

### General note on determining the shade with the DSA system

The DSA results greatly depend on the determination of the three reference teeth and optimal picture taking.

The three reference teeth should be determined prior to the preparation quickly at daylight or under a daylight lamp. Colours in the vicinity can falsify the shade determination, which is why an unobtrusive environment should be ensured.

### What needs to be given particular attention during picture taking?

- The picture quality determines the value of the DSA results. Preferably use the autofocus and macro functions when taking the picture.
- Make sure to take a close-up picture centred on the incisal teeth, the three shade guide teeth and the markings on the DSA holder.
- Prevent reflection in the centre of the shade guide teeth and the incisors by good illumination and accurate positioning of the objects (do not use standard camera flashes).
- The inscription of the shade guide teeth must be complete and well recognizable (particularly important if a polarizing filter is used).

### What are suitable camera types?

Basically, all current digital cameras and high-quality smartphone cameras can be used to take the pictures. Differences between pictures of different camera types and brands are normal and will be blanked by the DSA software during the comparison with the reference teeth.

### Cleaning the DSA holder

The DSA holder was fabricated of proven materials. It must be disinfected after use. Sterilizing the holder at a temperature of max. 134 °C/273 °F in the autoclave is recommended. For a possible surface disinfection, cleaning agents containing aromatic compounds, ketones and ether should not be used. Please note that the holder is not monomer resistant and not UV resistant. Therefore, intense direct sunlight must be prevented.

**For further information on the Digital Shade Assistant system, please refer to the Programat Operating Instructions.**

## Examples of incorrect images:

### Errors

- Reference teeth twisted and not parallel to each other
- Reference teeth overlapping
- Reference teeth not entirely inserted into the DSA holder
- Markings  $\diamond$  and I cut off at the bottom
- Reflection on the metal holders

### Errors

- Reference tooth orientation "incisal edge to incisal edge" and "incisal edge to tooth neck" used for the same image
- Reference teeth twisted and not parallel to each other
- Reflection on the metal holders
- Markings  $\diamond$  and I cut off at the bottom
- Reference teeth not entirely inserted into the DSA holder

### Errors

- Incorrect image section selected
- Only two reference teeth rather than three
- Labelling of the reference teeth and markings  $\diamond$  and I are missing
- Teeth of the patient concealed by the reference teeth

**Such faulty images cannot be interpreted by the software.  
An error message appears in the furnace display.**

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## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**