Description

VivaStyle is a ready-for-use whitening gel for lightening discoloured teeth. The product is administered by the user. The gel is applied by means of a custom-fitted tray.

Composition

Glycerin, Urea (Carbamide) Peroxide, Carbomer, Tromethamine, Potassium Nitrate, Disodium Pyrophosphate, PEG-40 Hydrogenated Castor Oil, Sodium Stannate, Dipotassium EDTA, Aroma.

Active ingredients: 10% or 16% carbamide peroxide, which is equivalent to 3.6% and 5.8% hydrogen peroxide respectively.

One syringe contains 3.7 g gel. This corresponds to 3 ml.

Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Contraindication

VivaStyle Gel should not be used

- during pregnancy or lactation;
- by children;
- by heavy smokers;
- by people who consume large quantities of alcohol;
- by people who are known to be allergic to any of the material's ingredients;
- in the presence of cervically exposed teeth;
- in the presence of untreated primary or secondary caries;
- in the presence of restorations with marginal gaps.

Applicable to EU countries:

Contains carbamide peroxide. Avoid contact with eyes. Rinse immediately if product comes into contact with them. Not to be used on a person under 18 years of age. To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use. Keep out of the reach of children.

Side effects

Temporary sensitivity of the oral mucous membrane or the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

Application

- A dental professional should examine the patient's medical history, establish the indication and determine the duration of the treatment. It is advisable for the dental professional to monitor the whitening results. VivaStyle 10% or VivaStyle 16% is used by the patient at home as instructed by the dental professional.
- The first application of any tooth whitening cycle should be performed by dentists or under their supervision, provided the same level of safety is guaranteed. Subsequently, the gel must be placed at the disposal of the patient to complete the bleaching cycle at home.

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- VivaStyle should be used before new tooth-coloured restorations are placed, since restorative materials cannot be whitened. The treatment with VivaStyle should be completed two weeks prior to the placement of tooth-coloured restoration. Existing restorations may not match the appearance of the tooth structure after the whitening process and therefore may have to be replaced.

Step-by-step instructions

Pretreatment

Before the VivaStyle tooth whitening gel is used, the dental professional should carefully examine the patient's medical history and clean the patient's teeth professionally.

Tray fabrication

A properly fabricated tray will enhance the effectiveness of the whitening process. Well defined trays will improve patient comfort and reduce the chance of irritating the mucous membrane.

1. Take an impression and fabricate the stone model.
2. Heat the tray material on a vacuum former until it sags 1 to 2 cm. Adapt it to the model and place it under vacuum for 30 seconds.
3. After cooling, remove the tray from the model and remove excess material.
4. Carefully trim the tray about 0.5–1 mm short of the gingival margins, using either fine scissors or a blade.
5. Place the tray back on the stone model to assess the fit. Final shaping and defining of the periphery may be accomplished with a coarse acrylic polisher.

Instructing the patient

Careful instruction of the patient is fundamental to the safe and effective application of VivaStyle tooth whitening gel at home.

1. Review each individual step of the application procedure in detail with your patient, using the enclosed information on VivaStyle.
2. Make sure that the tray fits accurately.
3. Using a VivaStyle gel syringe, show the patient how to fill the tray correctly. Important: Do not overfill the trays and only apply gel on the surfaces where it is needed. The calibration markings on the syringe should be used as a guide when extruding the material.
4. Show the patient how to insert the tray into the mouth. Explain the proper procedure for removing any excess tooth whitening gel.
5. Determine how long the patient should wear the tray. One hour a day is usually sufficient. Depending on the individual requirements of the patient, a longer wear time may be prescribed. The overall application time should not exceed four to six weeks.
6. Explain the proper removal and cleaning procedures for the tray. The tray should be stored in the accompanying case.

Notes

- The patient should be provided with the correct number of syringes needed for the treatment.
- Patients should be informed about the following points in particular:
 - They should avoid the consumption of citrus fruit and fruit juices during the treatment, as they can promote sensitivity.
 - They should try to avoid, or at least minimize, the consumption of coffee, tea, red wine and colas as well as smoking during the treatment to prevent the risk of staining the teeth.
 - Overfilling the tray with tooth whitening gel does not result in faster or better results. Instead, it may promote sensitivity.
 - Excess gel should be carefully removed to prevent irritation of the gums.
- Patients should call their dentist if they have any questions about the treatment or if unpleasant sensitivity occurs.

Warning

Avoid contact of the gel with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

Storage

- Protect VivaStyle syringes from heat and sunlight
- Do not use after the date of expiration
- Store at 2 – 28 °C / 36 – 82 °F.

Keep out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia
Tel. +61 3 9795 9599, Fax +61 3 9795 9645

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai Trading Co., Ltd.

2/F Building 1, 881 Wuding Road, Jing An District, 200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, 74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, 73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, Andheri (West), Mumbai, 400 053, India

Tel. +91 22 2673 0302, Fax +91 22 2673 0301

www.ivoclarvivadent.in

Ivoclar Vivadent Marketing Ltd.

The Icon, Horizon Broadway BSD, Block M5 No. 1

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawła II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent LLC

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

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www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Compass Building, Feldspar Close, Warrens Business Park

Enderby, Leicester LE19 4SD, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

www.ivoclarvivadent.us

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Ivoclar Vivadent AB